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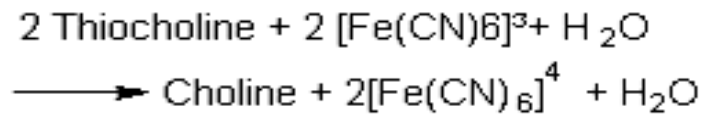
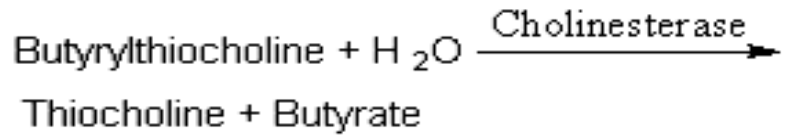
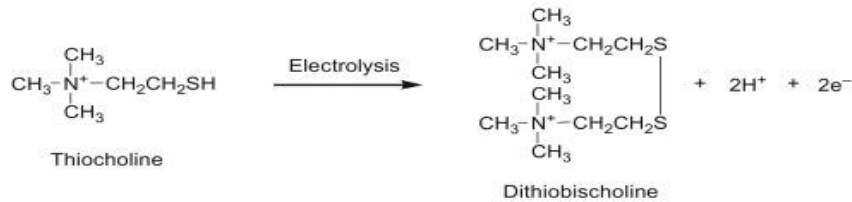
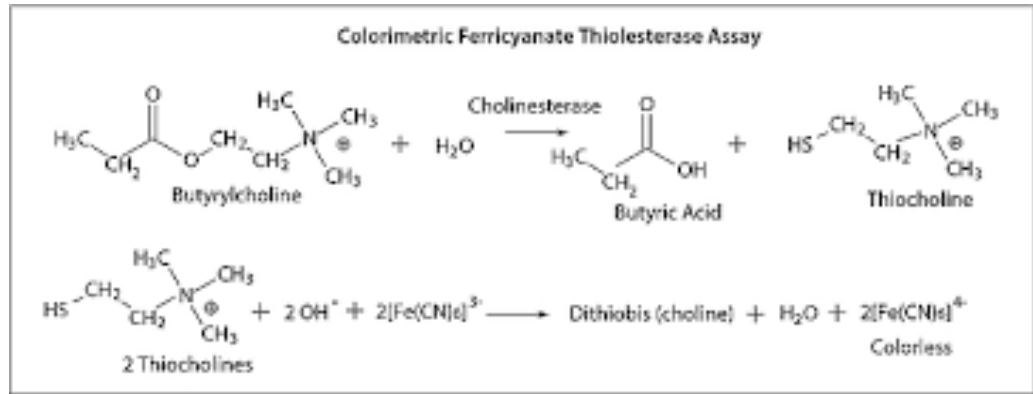
### Reagent 2:R2

Butyrylthiocholine ≤ 18 mmol/L

**Sensitivity (LOD):** The lower limit of detection is 40 U/L

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## SAMPLE

Serum and heparin plasma are collected by standard procedures.

Specimens should be protected from direct exposure to light. Samples are stable for 15 days at 2-8°C.

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## REAGENT PREPARATION

Working reagents are stable at 2-8°C in case of closed vials and avoiding contamination after preparation.

For manual working procedures; if working reagent will be used; shake the Reagent 2 vial gently before pouring its contents into the Reagent 1 bottle. It is advisable to wash the Reagent 2 vial with a small volume of the prepared mixture in order

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## CALCULATION

$$\Delta A / \text{Min} \times 65800 = \text{Che Activity in U/L}$$

## PERFORMANCE CHARACTERISTICS

The test has been developed to determine CHE activities.

**Low linearity:** 50 U/L

**High Linearity:** The method is linear up to up to 25000 U/L.

If these values are exceeded the sample should be diluted 1+5 with NaCl solution (9 g/L) and results multiplied by 6.

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Men 4620 U/L 11500 U/L

## NOTES

1. For in vitro diagnostic use only. Do not pipette by mouth. Avoid contact with skin and mucous membranes.
2. All the calibrators and controls must be considered as human & animal sample, so potentially infectious; all the protection actions must be applied to avoid any potential biological risk.
3. Material safety data sheet will be supplied on request.
4. Exercise the normal precautions required for handling laboratory reagents.
5. Caps of the reagents bottles cannot be used between two different kind of reagent and between R1&R2.
6. Reagents with different lot numbers should not be interchanged or mixed.

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